



## Premium Magnifying Lamp with Stand



Item #390411 • New Hudson, MI 48165  
www.tngworldwide.com • 800.520.6000  
Made in China • ©2018 TNG


Congratulations on your purchase of the ForPro Premium Magnifying Lamp with Stand. Please read the instructions and safety tips before use. Your Magnifying Lamp comes with a one-year limited warranty. If for any reason you need service, please call 800-520-6000.

**Function:**

The light pipe produces incandescent light, which is illuminated to the skin and magnified via convex lens so that the pore tissue on the skin is seen clearly. Beautician may use the magnifier to shape eyebrows and remove scars and birthmarks and other discolorations.

**Overall construction:**

- A: Power switch
- B: Light base
- C: Movable arm
- D: Adjustable triangular screw
- E: Power supply box

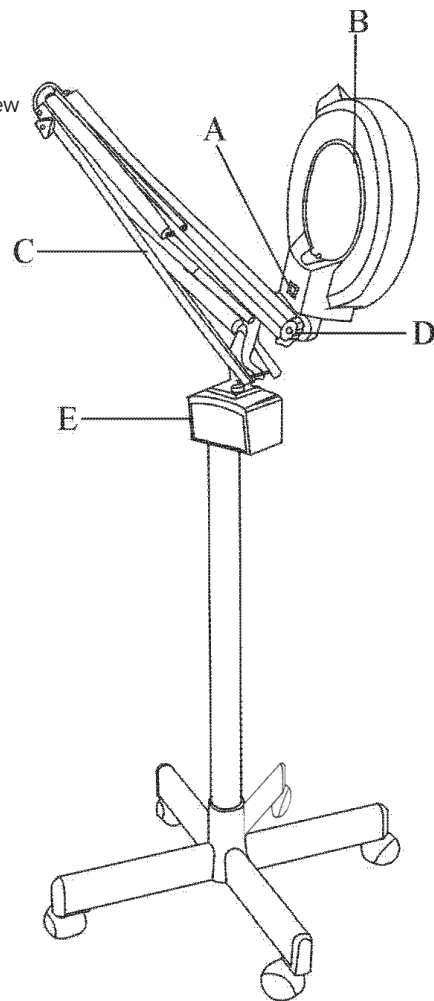

**Operation directions and procedures:**

- a) Assemble the stand and the return pipe.
- b) Insert the luminescent magnifier into the return pipe.
- c) Loosen three triangular screws on the movable arm and adjust the movable arm to the desired angle.
- d) Turn the light base to the desired position. Press the power switch to turn on power.

**Precautions:**

- a) Gently adjust the movable arm.
- b) The light base can only be turned up to 180°, otherwise the power cord may be damaged.
- c) If the power cord is damaged, replace it with a power cover cord of the same kind or refer to a professional technician for repair.

**Instructions:**

- a) Rating Voltage: 110V/50Hz.
- b) Rating Power: 10W
- c) Model: Premium Magnifying Lamp with Stand

**Warnings:**

- 1. This machine is not suited for the use by those with physical weaknesses or slow reactions (including children) unless guided by an individual capable of safe usage.
- 2. Keep away from children.
- 3. Keep away from water or outdoors.